Slovensky

Ceram•X™

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Insert Compules tip of chosen shade into the applicator gun barrel. Be certain that the collar on the Compules tip is inserted first. Remove the cap from the Compules tip. The Compules tip may be rotated to gain the proper angle of entrance into the cavity. To dispense the material, use a slow, steady pressure. Excessive force is not necessary. Dispense Ceram·X directly into the cavity.

Použitie striekačiek Easy-Twist

Dispense the necessary amount of Ceram·X restorative material from the Syringe onto a mixing pad by turning the handle slowly in a clockwise direction. To prevent oozing of the material when dispensing is completed, point the front tip of the syringe upwards and turn the handle counter-clockwise. Immediately re-close the syringe with the respective cap. Place Ceram·X in increments into the cavity and protect remaining material against light.

Rebuilding the tooth following either Single or Double Translucency Technique

Prior to the application of Ceram·X *mono* or *duo* shades, a flowable restorative material (e.g. X-flow™) can be used for e.g. cavity lining. For application please follow the directions for use of X-flow.

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Fill the complete defect with the chosen Ceram·X *mono* shade (with reference to incremental polymerization technique).

Ceram·X *duo* (systém dvojitej translucencie)

Rebuild the tooth with respect to the anatomical conditions. First rebuild a dentin core with the suitable Ceram·X *duo* dentin shade. In a second step, add an enamel layer with the suitable Ceram·X *duo* enamel shade. It is advisable to apply Ceram·X *duo* enamels in a comparably thin layer, not exceeding the thickness of natural enamel. In the anterior area a silicon matrix (prefabricated under assistance of a mock-up) can facilitate the establishment of the anatomical form.

In any case incremental placement (in 2 mm layers or less) is recommended to minimize polymerization shrinkage. Light cure each increment according to the below table⁴:

	≥ 500 mW/cm ²	≥ 800 mW/cm ²
Ceram·X <i>mono</i> odtiene (M1 to M7)	20 sec	20 sec
Ceram·X <i>duo</i> dentínové odtiene (D1 až D4 a DB)	40 sec	30 sec
Ceram·X <i>duo</i> sklovinné odtiene (E1 až E3)	10 sec	10 sec

When using a LED curing device, curing time according to manufacture's instructions should be applied.⁵

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Begin finishing immediately after final curing. For removal of excess and contouring finishing burs or diamonds may be used.

Additional finishing and polishing is achieved by using ENHANCE® Finishing Discs, Cups or Points. Alternatively, other standard aluminium oxide disc series may be used. Prior to proceeding to the use of polishing Discs, Cups or Points restoration surface should be

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finished to a final outline form and desired contour and anatomical features. Surface should be smooth and defect free.

For achieving a very high luster of the restoration use PoGo® Polishers or Prisma®Gloss™ regular paste followed by PrismaGloss Extrafine polishing paste with Enhance polishing foam cups.

MAINTENANCE OF the Compules Tip Applicator Gun

The applicator gun is sterilizable by autoclave (2.1-2.4 bar/135-138°C). Alternatively, suitable disinfecting solutions can be applied following the manufacturers' instructions. Do not continue use of damaged and/or soiled applicator gun. It is recommended that the applicator gun be disassembled for assured sterilization. Partially close the applicator gun and place thumb under the rear portion of the hinge. Push upward and lift hinge separating the applicator gun, exposing the plunger. Remove residual composite with a soft paper tissue and a suitable solvent (70 % alcohol). To reassemble, insert plunger into applicator gun barrel, press components together and snap hinge mechanism in place.

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Vybalené kompule a striekačky

In order to benefit from optimal handling properties, storage of unsealed Compules tips or syringes in a dry environment (< 80% relative humidity) protected from sunlight and used within 3 months is recommended.

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© DENTSPLY DETREY, 16. januára 2009
**DENTSPLY DETREY GmbH (Výrobca), De-Trey-Strasse 1, 78467 Konstanz, Germany